**EAR AND FOREHEAD  
INFRARED THERMOMETER  
FT-F21**

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# IMPORTANT! KEEP FOR FUTURE REFERENCE! READ CAREFULLY!

## CONTENTS

INTRODUCTION .....	3
IMPORTANT CLEANING INFORMATION.....	3
CARE AND CLEANING .....	4
IMPORTANT SAFETY INFORMATION .....	4-6
BODY TEMPERATURE .....	6
PACKAGE COMPONENTS .....	6
NORMALIZED SYMBOLS .....	6
PRESENTATION .....	7
REPLACING THE BATTERY .....	8
BEFORE USE .....	8
SOUND OPTION .....	9
CELSIUS / FAHRENHEIT OPTION .....	9
HOW TO CHANGE FOREHEAD / EAR TEMPERATURE MODE .....	9
CHECK THE MEMORY .....	10
HOW TO MEASURE FOREHEAD TEMPERATURE .....	10
HOW TO MEASURE EAR TEMPERATURE .....	11
TEMPERATURE TAKING TIPS .....	12
TROUBLESHOOTING .....	12
SPECIFICATIONS .....	13
DISPOSAL .....	13
STATEMENTS AND DECLARATIONS .....	14
IMPORTANT INFORMATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC) .....	14


## INTRODUCTION

This thermometer has been carefully developed for accurate, safe, and fast human body temperature measurements through the ear and the forehead.

This thermometer uses an offset when taking forehead and ear measurements so that the reading reflects the body's core temperature.

When used in accordance with the user's manual, this thermometer can be easily maintained.


Please note that the ear is an ideal site for taking a body temperature measurement. This is because body temperature is regulated by the hypothalamus, which shares the same blood supply as the tympanic (ear) membrane.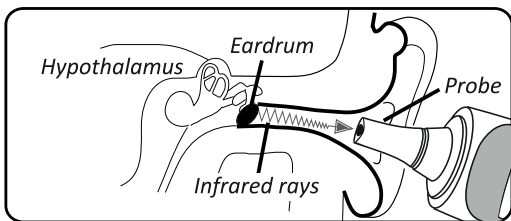

Forehead temperature is the same temperature as the arterial blood supply under the skin. It is the good determinate of body temperature, and unaffected by the artificial errors and time delays of oral and rectal methods.

## IMPORTANT CLEANING INFORMATION

- Earwax in the ear canal and lipid, oil, or sweat on the forehead may cause an inaccurate measurement. Please make sure that the ear canal and forehead is clean before attempting a measurement.
- If you clean the ear or forehead, please wait for 5-10 minutes before measuring.
- Don't touch or blow at the infrared sensor. A polluted or contaminated infrared sensor may cause inaccuracy. If you think you have contaminated the probe sensor, please clean it.
- To clean the sensor, gently wipe its surface with a cotton swab slightly moistened with alcohol and immediately wipe dry with a clean cotton swab. After cleaning, allow at least 20 minutes of drying time before taking measurements.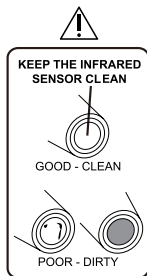

## CARE AND CLEANING


To ensure accurate measurements, it is very important to keep the probe tip clean and free of scratches. Fingerprints, earwax or dirt will affect the accuracy of the thermometer. In order to get an accurate measurement, please clean the probe tip.

Gently wipe its surface with a cotton swab slightly moistened with 70-90% alcohol and immediately wipe dry with a clean cotton swab.

Use a soft dry cloth to clean the body of the thermometer. Never clean the thermometer with an abrasive cleanser, thinner, benzene or submerge the thermometer into water or other liquids.

After cleaning, place the thermometer in the protective case. Store it in a clean, dry place at room temperature. Never expose the thermometer to extreme temperatures, humidity, direct sunlight or shock.

**WARNING!** Do not service and maintain while the device is in use.

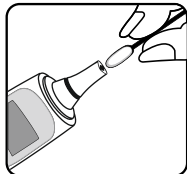## IMPORTANT SAFETY INFORMATION

This device is not a life supporting me equipment.

Proper technique is critical in order to obtain accurate temperatures. Therefore, in order to obtain an accurate measurement, please read this manual carefully before using this thermometer.

- Use of this thermometer is not intended to substitute professional consultation by your physician. Measurement results are for reference only. Contact your physician about any known or suspected health concerns.
- The Infrared sensor must be kept clean, dry, and undamaged at all times to ensure accurate measurements.
- Do not use the ear/forehead measurement mode for measurements other than for human body temperature.
- Avoid measuring the ear temperature if the ear is inflamed or infected.
- If the temperature of the storage area differs greatly from that of the measuring area, please allow time for the thermometer to equalize to the current environment's temperature. For great differences in environmental temperatures, let the thermometer equalize for 30 minutes before use.
- The patient is an intended operator. The patient can measure, transmit data under normal circumstances and maintain the device and its accessories according to the user manual.

- Keep the thermometer out of the reach of children and pets to avoid inhalation or swallowing of small parts. Do not allow children to take their temperatures unattended. Children may not be able to use the device according to the instructions in this user manual. This thermometer is not a toy.
- Don't store the thermometer in extreme temperature environments (below -13°F /-25°C or over 131°F/55°C) or in extreme humidity (Rh≥95% ).  
The thermometer in these extreme environments may cause inaccuracy.
- Don't use this product in ear mode inside a wet ear, such as after showering, swimming, or bathing. Doing so may hurt the ear or damage the thermometer.
- Don't use the thermometer if it is damaged in any way (for example, the infrared sensor is broken). The continuous use of a damaged unit may cause injury, improper results, or serious danger.
- Don't take apart, repair or change any parts of the unit at any time outside of replacing the battery.
- Don't use a mobile phone nearby when the thermometer is in use.
- If your thermometer will not be used on a regular basis, remove the battery to prevent possible damage due to chemical leakage. If the battery does leak, remove carefully.
- If there is any fluid leaking from the battery, protect your bare skin. When using, do not touch the patient and the battery simultaneously.
- This device will not cause toxicity, allergies or irritation from normal use. However, if you have an allergic reaction to any material, please stop the use of this thermometer immediately and consult your physician.
- Store the thermometer in a dry, clean place. Do not leave the product exposed to any chemical solvent, lint, dust, direct sunshine or high temperatures.
- Do not put the IR Thermometer with keen-edged goods together.
- Please do not place the thermometer probe on scarred tissue, tissue compromised by skin disorders, or use on trauma patients.
- It is not advisable to use the thermometer on patients that are being treated with certain skin anti-inflammatory drugs, to place the thermometer probe on skin exposed to direct sunlight, fireplace heat, to air flow from air conditioners, and to use on patients undergoing cold compress therapy.
- If the following conditions occur, we recommend you to take your temperature at least three times and use the highest reading.
  - Infants under three months old.
  - Children under three years old who have a compromised immune system.
  - This is your first time using the IR thermometer.
- Not intended to be sterilized. Not for use in an OXYGEN RICH ENVIRONMENT !

- Please report, if any unexpected operation or events occur.
- After drop, shock or other events, that may cause changes in the performance, please contact with the service personnel.
- This device must only be serviced, repaired and opened by individuals at authorized services .

## BODY TEMPERATURE

Normal body temperature has a range. A person's normal temperature range tends to decrease with age. The following table shows normal temperature ranges by age:

0 – 2 years	36.4 – 38.0 °C	97.5 – 100.4 °F
3 – 10 years	36.1 – 37.8 °C	97.0 – 100.0 °F
11 – 65 years	35.9 – 37.6 °C	96.6 – 99.7 °F
> 65 years	35.8 – 37.5 °C	96.4 – 99.5 °F

The range of what is normal varies from person to person and can be influenced by many factors, such as time of day, measuring from different sites, level of activity, medications, emotion, and so on.

It is recommended that you practice with the thermometer on yourself and family members when you are healthy. This way you know how the thermometer works and will feel more confident with the measurements when the same person is ill.

## PACKAGE COMPONENTS

DESCRIPTION	QUANTITY
• IR thermometer	1
• Manual	1
• Storage case	1
• Batteries 2 X1.5V AAA	(Not included)

## NORMALIZED SYMBOLS

Symbol for "ENVIRONMENT PROTECTION – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local Authority or retailer for recycling advice"

Symbol for "CE"

Symbol for " RF transmitters"

Applied part of type BF  
(The applied part is the probe tip)

Refer to Instruction Manual.

Manufacturer


**IP22**

Symbol for "the IP classification"


European Authorized  
Representative

## PRESENTATION

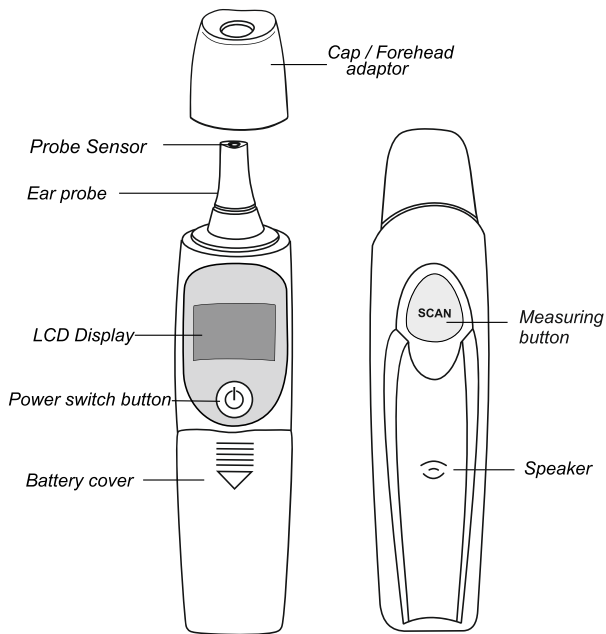## LCD DISPLAY
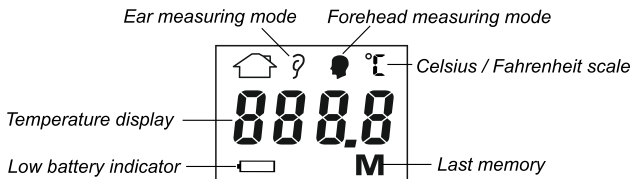
## INSERT / REPLACE THE BATTERY

When the low battery indicator “” appears on the LCD, or the thermometer does not function at all, you should replace the batteries as soon as possible.



Replacing the battery:

1. Remove the battery cover.
2. Insert the batteries as indicated inside of the battery compartment.
3. Replace the battery cover.
4. Dispose of used batteries according to the manufacturer's instructions.

### WARNING:

In some special conditions from the battery may leak liquids, which may cause a chemical burn or damage the product. If the battery leaks and comes into contact with the skin or eyes, wash immediately with copious amounts of water. Batteries should be handled under adult supervision.


- Do not cause short circuit of the battery terminals.
- Dispose of used batteries with care.
- Do not disassemble, recharge, or dispose of batteries in fire. The battery is likely to explode in fire.
- Do not mix old and new batteries or different types of batteries.
- It is better to use alkaline batteries.
- Keep the batteries out of the reach of children.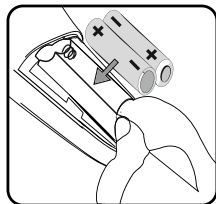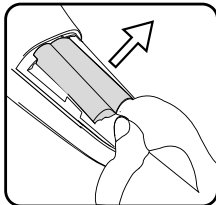

## BEFORE USE



1. To achieve accurate measurements, it is very important to check that the infrared sensor is clean before each use.

To clean the sensor, gently wipe its surface with a cotton swab slightly moistened with alcohol and immediately wipe dry with a clean cotton swab. After cleaning, allow at least 20 minutes of drying time before taking temperatures.

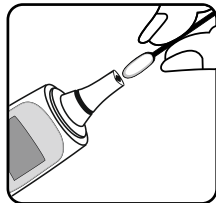


## SOUND OPTION

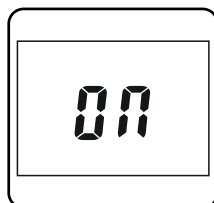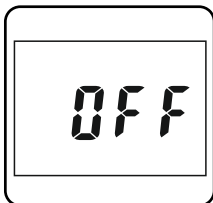While the thermometer is powered off, press and hold the "SCAN" button for about 5 seconds until the displayed screen appears On or Off. Release the button.

Then press briefly the "SCAN" button again and change the mode. For confirmation of the desired settings wait 2 sec. until the display turns off.




## CELSIUS / FAHRENHEIT OPTION

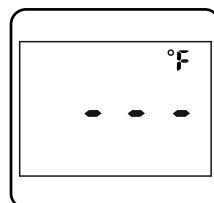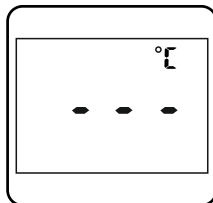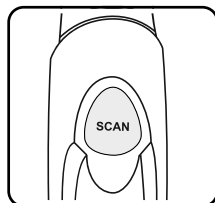While the thermometer is powered off, press and hold the "SCAN" button for about 8 seconds until the displayed screen appears °C or °F. Release the button.

Then press briefly the "SCAN" button again and change the Celsius scale or Fahrenheit Scale. For confirmation of the desired settings wait 2 sec. until the display turns off.

## HOW TO CHANGE FOREHEAD / EAR TEMPERATURE MODE

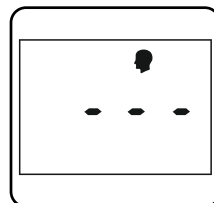
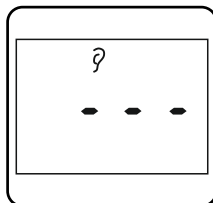
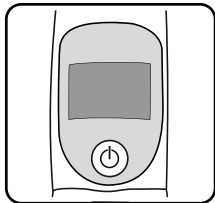
While the thermometer is powered off, press and hold the "Power switch button" for about 5 seconds until the displayed screen appears - - - with blinking ? or head silhouette symbols. Release the button.

Then press briefly the "SCAN" button and change the forehead / ear mode. For confirmation of the desired settings wait 2 sec. until the display turns off.

## CHECK THE MEMORY

This thermometer can recall the past 12 measurements.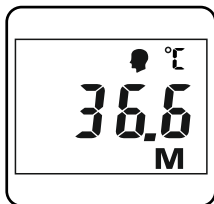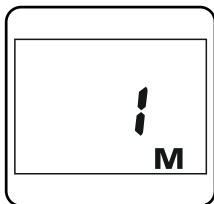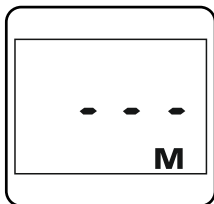

While the thermometer is powered off, press briefly [SCAN] button, then displayed screen appears --- with blinking M.


Press [SCAN] repeatedly to cycle through the 12 last saved temperatures in the memory. To exit the mode there are 2 options:

- 1- Press the and the thermometer will turn off
- 2- Do nothing 10 sec. the thermometer will turn off

## HOW TO MEASURE FOREHEAD TEMPERATURE


Set up the thermometer on forehead measuring mode with the desired scale °C or °F.

①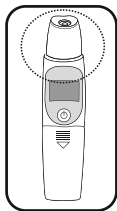


Use the thermometer with installed forehead adaptor.

②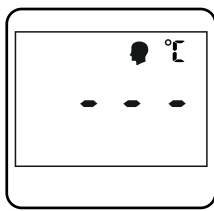


While the thermometer is powered off, press briefly "Power switch button" and the displayed screen should be as shown in the picture above with blinking °C or °F.

③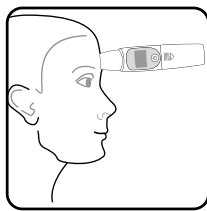

Hold the thermometer so that the forehead adaptor is aligned with the centre of the forehead, midway between the eyebrow and the hairline. Once the device is touching the forehead as indicated, hold the thermometer steady and press "SCAN" to start the measuring. Keep the thermometer flat until you hear a beep. (if the sound feature is turned on) When the measurement is complete, the thermometer may be removed from the head.

④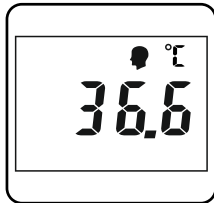

Read the temperature on the display.

## TEMPERATURE TAKING TIPS

1. A temperature measurement taken in the right ear may differ from a measurement taken in the left ear. Therefore, always measure the temperature in the same ear.
2. External factors may influence the ear and forehead temperatures, particularly when an individual has:
  - been lying on one ear or the other
  - had their ears covered
  - been exposed to very hot or very cold temperatures, or have recently been swimming or bathing.
 In all of the above cases, wait 30 minutes before taking measurements.
3. For persons wearing hearing aids or earplugs, remove such devices and wait 30 minutes using the device in ear mode.
4. If ear drops or other topical ear medications are being used, use the untreated ear for measurements. If both ears are receiving treatment, please use the device in forehead mode.


## TROUBLESHOOTING

Problem	Checklists	Countermeasures
No response	Battery dead ? Incorrect polarity ? Poor battery contact ?	Change batteries. Take out the battery, and reinsert it correctly.

Description	Meaning
H <sub>i</sub>	Measuring temperature over 43.0°C/109.4°F
L <sub>o</sub>	Measuring temperature lower than 34.0°C/93.2°F
Err	Environment temperature higher than Working temperature. 10.0°C- 40.0°C(50.0°F- 104.0°F)


## HOW TO MEASURE EAR TEMPERATURE

Set up the thermometer on ear measuring mode with desired scale °C or °F.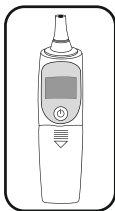


①

Use the thermometer with removed forehead adaptor.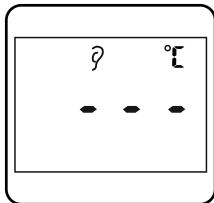


②

While the thermometer is powered off, press briefly "Power switch button" and the displayed screen should be as shown in the picture with blinking °C or °F.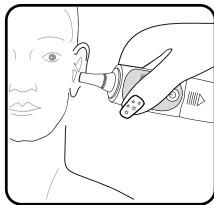

③

Insert the probe carefully into the ear canal. Hold the thermometer so that the probe is going straight into the ear toward the eardrum. It is important that the probe sensor faces the eardrum. Hold the thermometer steady and press "SCAN" to start the measurement. Keep the probe still until you hear a beep (if the sound feature is turned on). When the measurement is complete, the thermometer may be removed from the ear.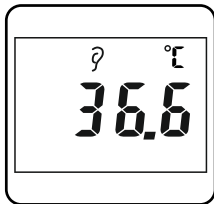

④

Read the temperature on the display.

## SPECIFICATIONS

Model	FT-F21
Measuring Range	Body 34.0°C-43.0°C( 93.2°F-109.4°F)
Measuring time	minimum measuring time is 2 seconds; and minimum time between measurements for the same site is 2 seconds
Accuracy	±0.2°C/0.4°F    34.0°C - 34.9°C    (93.2°F - 94.82°F) ±0.2°C/0.4°F    35.0°C - 42.0°C    (95.0°F-107.6°F) ±0.2°C/0.4°F    42.1°C - 43.0°C    (107.78°F-109.4°F)
Resolution	0.1°C/0.2°F
Operation Condition	10.0°C-40.0°C(50.0°F- 104.0°F) Rh≤95% Pressure: 86~106 kPa
Storage Condition	-25.0°C-55.0°C(-13.0°F- 131.0°F) Rh≤95% Pressure: 86~106 kPa
Battery type and battery life	AAA 1.5V 2pcs(FT-F21) The typical service life of the new and unused batteries is 300 measurements for the operation time is 60s
Protection against electric shock	Internally powered ME equipment
Classification:	Type BF  applied part
IP classification	IP22
Mode of operation	Continuous operation
Size (body)	L 145*W 35*H 40 mm FT-F21
Weight (including battery)	80g (FT-F21),

Software Version: V1.6  
Manual Version: V1.6

## DISPOSAL

This product needs two 1,5V "AAA" batteries. Batteries contain substances that can pollute the environment. Do not dispose of batteries with ordinary household waste, and have them in specialized collection point for batteries. Always remove the batteries before disposing of the device or pass it in an official collection point. **IMPORTANT!** After stop using the product, do not throw it away with the common waste. Take it to a specialized point to be recycled. In this way you keep the environment safe.

## STATEMENTS AND DECLARATIONS:

- 1) This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.
- 2) Do not use a mobile phone or other devices that emit electromagnetic fields, near the unit. This may result in incorrect operation of the unit.
- 3) Caution: This unit has been thoroughly tested and inspected to assure proper performance and operation!
- 4) Caution: This thermometer should not be used adjacent to or stacked with other equipment. If said equipment must remain in close proximity to the thermometer being used, it should be monitored in order to verify normal operation during the time of measurement.

### Important information regarding Electro Magnetic Compatibility (EMC)

With the increased number of electronic devices such as PC's and mobile phones, medical devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and create a potentially unsafe situation. Medical devices should also not interfere with other devices. In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the EN60601-1-2:2007 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices. This medical device manufactured conforms to this EN60601-1-2:2007 standard for both immunity and emissions.

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# SGS

EC Certificate Production Quality Assurance System FI15/07012

The management system of

## Fudakang Industrial Co., Ltd

No. 8 Yinghe Road,  
Yuanjiangyuan Management Zone  
Changping Town, Dongguan, Guangdong,  
P.R. China

has been assessed and certified as meeting the requirements of

### Directive 93/42/EEC on Medical Devices, Annex V

For the following products


Electronic clinical thermometers,  
infrared clinical thermometers,  
electronic blood pressure monitors

Products covered are listed in Attachment 1 of this certificate

This certificate is valid from 18 June 2019 until 22 December 2023  
and remains valid subject to satisfactory surveillance audits.  
Issue 3. Certified since 22 December 2015  
This certification is based on decision: FI15/07012P4

Authorised by

**SEAL FOR THE COPY  
OF CERTIFICATE ONLY**

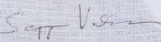  
Seppo Vahasalo, Notified Body Manager  
SGS Fimko Ltd., Notified Body 0598

SGS Fimko Ltd Särkinientie 3, PO Box 30 FI-00211 Helsinki, Finland  
t. +358 9 636 361 f. +358 9 692 5474 [www.fi.sgs.com](http://www.fi.sgs.com)

Business ID 0978338-5

Member of the SGS

Group (SGS SA)

# FUDAKANG INDUSTRIAL CO.,LTD

Address : No.8 Yinghe Road, Yuanjiangyuan Management Zone,  
Changping Town, Dongguan, Guangdong, China  
TEL:86-769-81098181 FAX:86-769-81098187  
E-mail: bobyu@fudakang.com sales@fudakang.com

## Declaration of RoHS Compliance

Company Name : Fudakang Industrial Co., LTD.

Company Model Number : FT-1021

Description : IR Thermometers

1. We hereby declare that the above model is compliant to the Restriction of Hazardous Substances in Electrical and Electronic Equipment Directive (RoHS) 2002/95/EC.
2. We hereby agree that only RoHS compliant materials and components will be used in the assembly of the above model. Under no circumstances will non-compliant materials and components be used in place of approved compliant equivalents during the life of the product.
3. Should it be necessary, we agree to supply documentation demonstrating compliance to the above directive on individual components or materials used in the assembly of the above model as and when requested by the DIDIS LTD BULGARIA. This will be in the form of a test report from an independent accredited test laboratory.

Signed with company chop:  
For and on behalf of  
FUDAKANG INDUSTRIAL CO., LTD.  
东莞市福达康实业有限公司

.....  
Authorized Signature(s)

Position: President

Date: 30, May 2018

Fantronic Composites,  
Rue Jacques  
Laurent ZA Sud-Est-BP 1185150  
La Mothe Achard France

Manufacturer  
FUDAKANG INDUSTRIAL CO.,LTD  
Address: No.8 Yinghe Road

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Didis LTD, Bulgaria