**BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА**  
**EN: MANUAL INSTRUCTION FOR USE**  
**DE: MANUELLE BEDIENUNGSANLEITUNG**  
**GR: ΟΔΗΓΙΕΣ ΧΕΙΡΙΣΜΟΥ ΧΕΙΡΙΣΜΟΥ**  
**NL: GEBRUIKSAANWIJZING**  
**RO: INSTRUCȚIUNI DE UTILIZARE**

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EN

Read carefully this manual before using the nose cleaner, to ensure the correctly and safe usage of the product. Keep the manual for future reference, because it contains important information. The product meets the requirements of standards: EN 55014-1; EN 55014-2; EN 61000-3-2; EN 61000-3-3

Dear Clients,

Thank you for choosing our Nose cleaner Purity - NC001. We hope it will be maximum useful for you and we recommend you to read this manual carefully before first usage of the product. The nose cleaner is an essential element for effective maintenance of the hygiene of the nose cavity.

Until it reach it's first year, your child breaths through the nose, this is why it is important to clean regularly the nose cavity of your baby. Harmful sludge can concentrate in nose canals as bacteria, microbes, viruses, dust and so on. Bad hygiene of the nose canals can lead to infections as cold and as a result can lead to more serious health problems. Little children learn to blow their nose after 3 years old. Nose clogging makes breathing hard, which can cause different problems with sleep and eating, therefore it is important to clean nose canals and to remove secretions regularly.

## SAFETY INSTRUCTIONS

1. The device is using only for secretion cleaning and mucus from your baby's nose.
2. Use only accessories and spare parts, ensured by the manufacturer.
3. Do not throw or drop the device, otherwise you can damage it.
4. Only an adult can work with the device. Keep away from children access.

5. Use only alkaline batteries. Do not use rechargeable batteries with this product.
6. Remove the exhausted batteries immediately. Change old batteries with new ones, as described in this manual. Keep the polarity marking in the battery compartment, according to the mark on the batteries.
7. When you do not use the product for long time, please take the batteries out.
8. Strictly follow this user manual and keep it for future reference.
9. Please, keep in mind that the working mode of the product is with interruptions. The recommended time for using is 5 minutes, after which the device must rests at least 10 minutes.
10. **ATTENTION!** The product is not a toy!
11. **WARNING!** This device is not intended for using by people (including children) with decrease physical, sensitive or mental abilities or lack of experience and knowledge, unless they are observed or taught how to use the device by person, which is responsible for their safety.

## BEFORE USAGE

1. Don't forget to sterilize the accessories before and after every usage of the product. Including the silicone nozzle and the transparent cover.
2. Clean your hands before and after every usage of the product, to avoid infection, if your child is sick.
3. Ensure that the silicone sealant is not damaged or worn out, pulling it out gently in all directions. If there are any signs of damage or weakness, stop using it and replace it immediately.
4. Do not immerse the device in water.
5. Do not reach the device, if it fell in water.

## PRODUCT CHARACTERISTICS

1. Convenience and hygiene and reliable secretion caps.
2. The maintenance of clean nose cavity decreases necessity of medicines.
3. Easy for exploitation.
4. Highly effectivity.
5. 3 suction levels.
6. 3 musical options

The nose cleaner is device, which helps for innovative, pure, effective and easy for maintenance health care.

## USAGE INSTRUCTIONS

1. Wash your hands before using the product.
2. Keep your baby's head up by the time of using the product, to ensure unhindered movement of the air and to favor your baby's breathing.
3. If your child has got cold, do not forget to use the nose cleaner before eating, to make your child's breathing easier.
4. Your baby's mouth must be opened by the time of using the product, to ensure unhindered air flow.
6. Exhausted batteries must be taken out from the device immediately.
7. Take the batteries out, if you will not use the product for long time.
8. Do not shorten the batteries' terminals.
9. Always keep the batteries dry. Do not allow to batteries to get in contact with fresh or salt water.

## DEVICE DESCRIPTION

Look page 2: A. Loudspeaker; B. Battery compartment cap; C. ON/OFF battery; D. Silicone sealant; E. Silicone nozzle; F. Cap; G. Hygienic secretion caps; H. Hygienic cap compartment; I. Air inlet; J. LCD display; K. Menu button; L. „+“ и „-“ setting button

## PRODUCT ASSEMBLING

Look page 3.

1. After placing the batteries (1) (when using the device for the first time), put the secretion cap in the intended compartment. (3,4).
2. Ensure that the silicone sealant is on the correct place and position (2), which allows to the nose cleaner to be sealed.
3. Put the cap (5) back on its place and turn, until the device gets closed.
4. Put the nozzle (6).

## INSUTRCIONS FOR WORKING WITH PRODUCT

Look page 2.

Turn the product on, as you press button „1“. Press „SET“ button and hold for 3 seconds, until „ASP“ shows on the display. Press „+“ or „-“ button, to choose power of level of suction (Low, middle, high). Press button „SET“ again and the display will show „MUS“. Use „+“ и „-“ button, to choose musical option from the menu. Press button „SET“ again and the display will show „VOL“. Press „+“ or „-“ button, to choose level of melody volume.

## TECHNICAL CHARACTERISTICS

Maximum suction power– 50 kPa

Do not block the air inlet, while using the device.

## ACCESSORIES, INCLUDED IN THE SET

3 size silicone nozzles, adapted to the nostrils of the baby in its different ages:

- Pink – 0-3 months

-Blue – 3-12 months

-Orange – 12 months up

-25 one-off hygienic secretion caps

-2 reserved silicone sealants

0-3 months

3-12 months

+ 12 months

## GENERAL TIPS FOR USE

1. Turn on the device through pressing the button for turning on.
  2. Put the nozzle in the baby's nostril carefully, until you keep baby's head up in straight position, for helping it's breathing.
  3. By the time of suction, ensure that the baby's mouth is opened, to ensure releasing of the internal ear pressure.
  4. Suction prolong: 8 second for nostril
- To avoid suction of baby's nose cavity wall, to prevent injuries.

- Do not try to insert the whole nozzle in the baby's nostril.
- Avoid sharp movements and excessive pressure , which can harm the baby's ear and sinuses. Repeat as many times as necessary to guarantee unstopping baby's nose. Change the hygienic caps, when it is necessary or after every usage of the product.
- When using saline, the liquid mix with baby's secretions and it can get into the bronchi. Ensure that the mixture is sucked out with the nose cleaner.

Menu	0			
Aspiration (ASP)	-	30% Low	70% Middle	100% High
Music (MUS)	OFF	Option 1	Option 2	Option 2
Sound volume (VOL)	-	Low	Middle	High

## BATTERY MANUAL

Alkaline batteries.

1. Alkaline batteries are not a toy. It is strictly forbidden children to play with them.
2. No attempts for charging of non rechargeable batteries should be made.
3. Do not mix old and new batteries from different: alkaline, standard or rechargeable.
4. Changing the batteries must be done by an adult.
5. Put the batteries in the battery compartment, while you take into account +/- polarity.
6. Exhausted batteries must be taken out from the device immediately.
7. Take out the batteries, if you will not use the product for long time.
8. Do not shorten battery terminals.
9. Always keep batteries dry. Do not allow batteries to get in contact with fresh or salt water.

## BATTERY CHANGING

1. Take off the battery compartment cap though pulling the lower part of the nose cleaner.
2. Put 2 AA batteries and ensure that the battery polarity coincides with polarity of the device.
3. Put the battery compartment cap.
4. Throw out the batteries in the intended places for battery disposal. Your batteries must be recycled or treatment by environmentally friendly way.
5. Do not throw the batteries with the household waste.
6. Do not throw the batteries in fire. The battery can explode or to leak.

close open

## TROUBLESHOOTING

PROBLEM	SOLUTION
The product do not suck	<ul style="list-style-type: none"> <li>-Check the closing mechanism (position, damage).</li> <li>-Check whether the device is well closed</li> </ul>

PROBLEM	SOLUTION
The product do not suck	<ul style="list-style-type: none"> <li>- Check the cover position</li> <li>- Check the position of silicone sealant.</li> <li>- Ensure that air inlet is not blocked</li> </ul>
Leaking or accumulation on water under the cover	<ul style="list-style-type: none"> <li>-Check whether the device is well closed.</li> <li>- Ensure that the cover is not in reverse</li> </ul>
Nothing happens, when you press the button for starting the device	<ul style="list-style-type: none"> <li>- Check the batteries (polarity, charge)</li> </ul>

Produced in CANGAROO in China

Importer: Moni Trade ltd.

Address: Bulgaria, Sofia, Trebich quarter, ul. Dolo 1

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**Importator: Moni Trade Ltd.**