BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
RO: INSTRUȚIUNI DE UTILIZARE
DE: GEBRAUCHSANWEISUNG
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
ES: INSTRUCCIONES DE USO
IT: ISTRUZIONI PER L'USO

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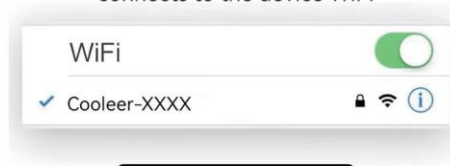
1


2


* Long press the power button to turn on, the phone connects to the device WIFI


СВЪРЗВАНЕ И ДОБАВЯНЕ НА УСТРОЙСТВО/ CONNECT AND ADD THE DEVICE/ CONECTAREA ȘI ADĂUGAREA DISPOZITIVULUI/ EIN GERÄT ANSCHLIESSEN UND HINZUFÜGEN/ ΣΥΝΔΕΣΗ ΚΑΙ ΠΡΟΣΘΗΚΗ ΣΥΣΚΕΥΗΣ/ CONECTAR Y AÑADIR UN DISPOSITIVO/ CONNESSIONE E AGGIUNTA DEL DISPOSITIVO/ CONNECTER ET AJOUTER L'APPAREIL/ ПОДКЛЮЧИТЕСЬ И ДОБАВЬТЕ УСТРОЙСТВО/ POVEZIVANJE I DODAVANJE UREĐAJA/ VERBINDING MAKEN EN HET TOESTEL TOEVOEGEN/ ESZKÖZ CSATLAKOZTATÁSA ÉS HOZZÁADÁSA/ PŘIPOJENÍ A PŘIDÁNÍ ZAŘÍZENÍ/ PRIPOJENIE A PRIDANIE ZARIADENIA

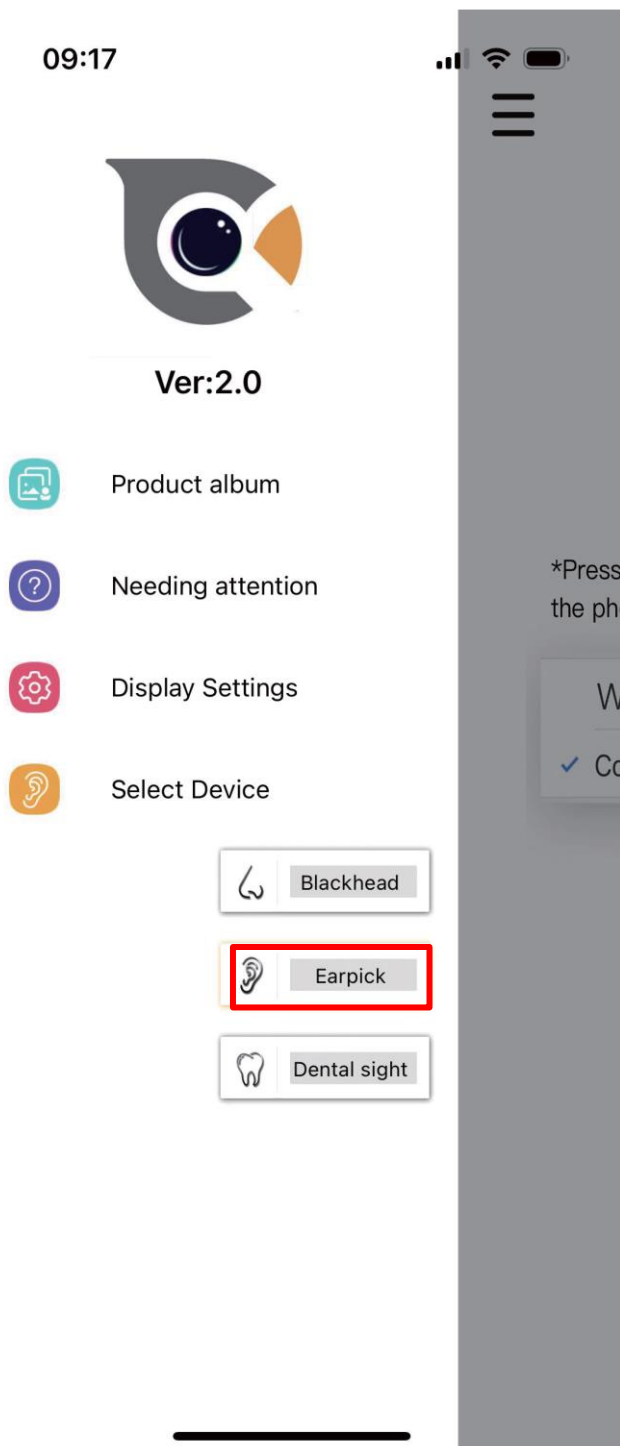
3

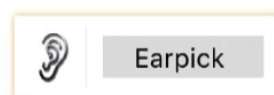4

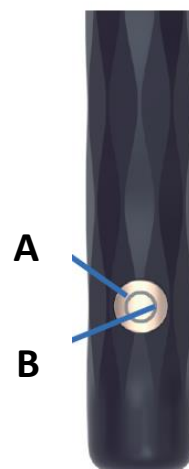5

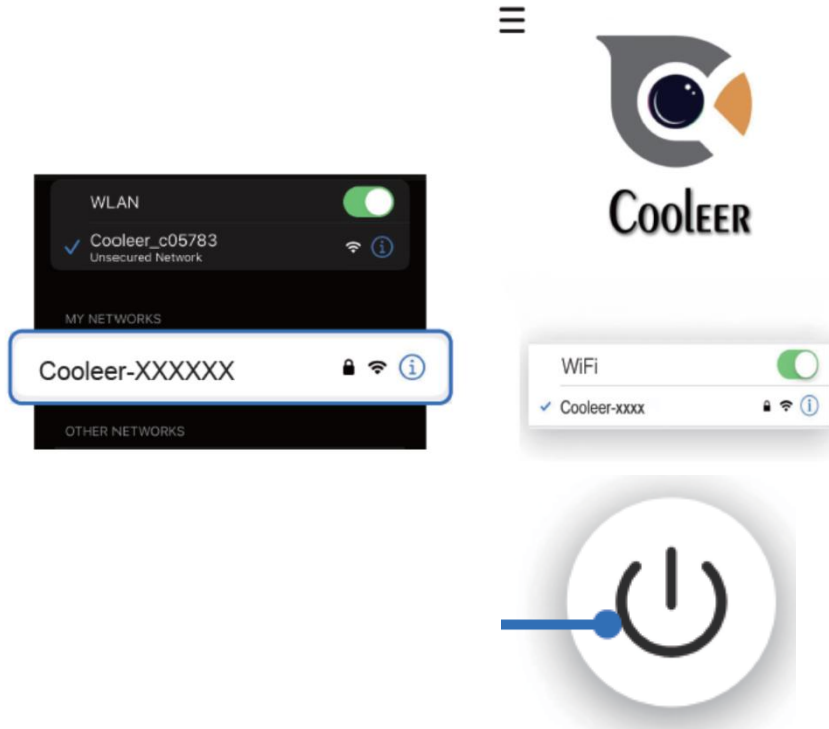6

To avoid affecting the normal use of the earwax removal kit, please be sure to follow the steps below to set it up.

6.1

Allow "Cooler" to use location?

Used to provide geographic location-based features or services

Allowed when using

Allowed for this use only

Reject

6.2

"Cooler-XXXXX" cannot access the Internet, can the Wi-Fi Assistant be allowed to switch to a better network?

Not allowed

Allow

6.1

Allow "Cooler" to use location?

Used to provide geographic location-based features or services

Allowed when using

Allowed for this use only

Reject

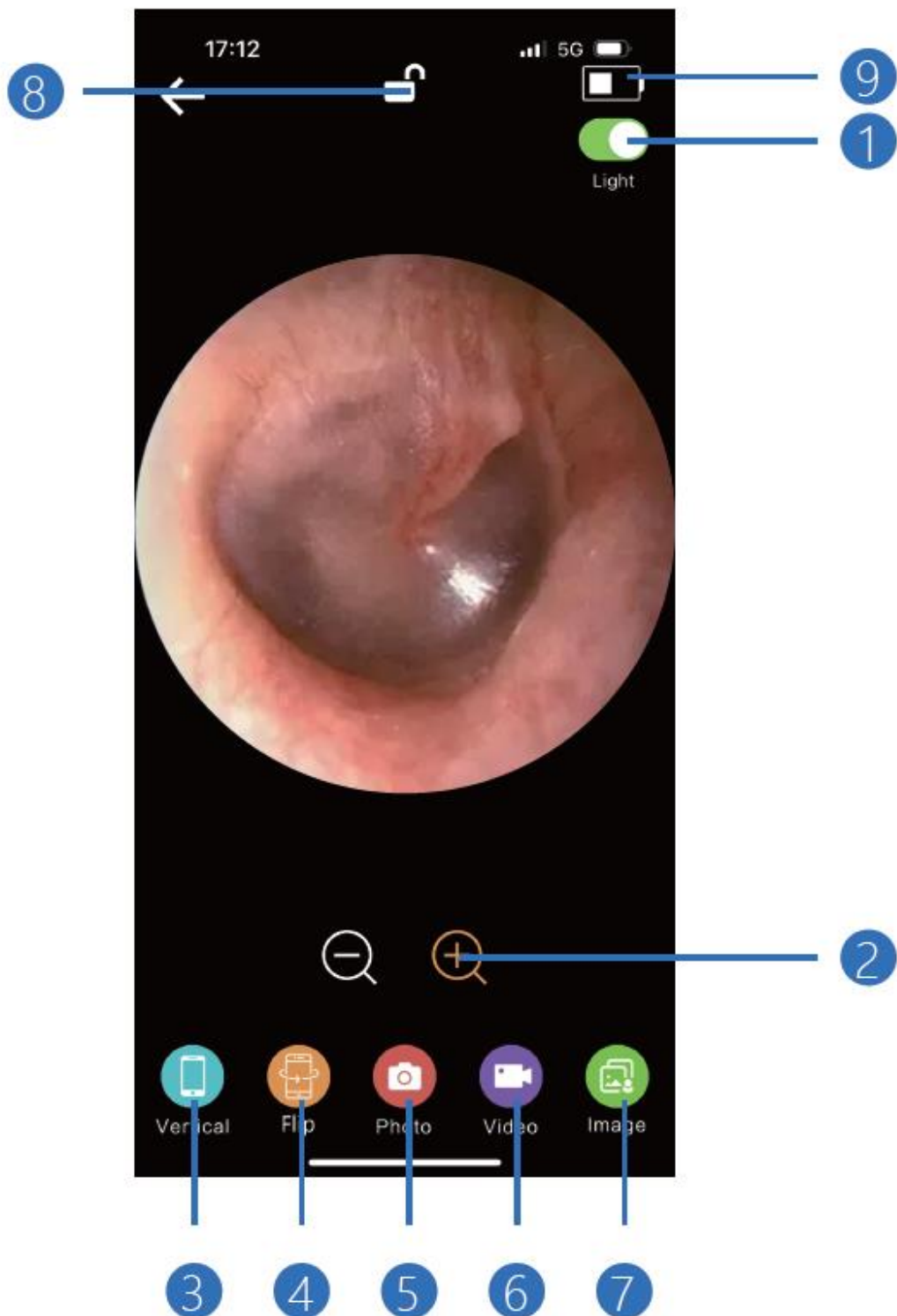
6.2

"Cooler-XXXXX" cannot access the Internet, can the Wi-Fi Assistant be allowed to switch to a better network?

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IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THE USER MANUAL CONTAINS IMPORTANT INFORMATION ON THE CORRECT USE AND MAINTENANCE OF THE PRODUCT.

WARNING! THE SCHEMES AND FIGURES IN THIS INSTRUCTION ARE FOR ILLUSTRATIVE PURPOSES ONLY AND THE PRODUCT YOU HAVE BOUGHT MAY DIFFER FROM THEM.

Cangaroo's digital wireless otoscope (OTO Y-201) is a multi-purpose device for effective and safe examination of the ear canals and gentle removal of earwax from the ears of children and adults.

•Using an HD camera with 3 MP resolution and a 3.5 mm lens diameter, the otoscope provides an image of the inside of the ear and allows easy viewing of the condition of the eardrum and the inside of the ear. The camera connects to your smartphone via a wireless Wi-Fi network and is controlled using the CooLeer mobile app, compatible with iOS and Android operating systems. It can also take pictures and record videos in real time.

•The device has a silicone ear scoop that is extremely gentle on the inside of the ear and prevents any injury during cleaning.

THE KIT INCLUDES (IMAGE P):

- **A - 8 differently shaped cleaning attachments** to accommodate different ages.
- **B - Small cleaning brush.**
- **C. and C.2 – 8 additional accessories** for easy and effective cleaning of earwax.
- **D – Otoscope with a built-in camera** - Compact size, portable and user-friendly.

GUIDELINES AND RECOMMENDATIONS FOR SAFE USE

WARNING!

- 1.The device must be used ONLY for the purposes, described in this manual.
- 2.The device is constructed for practical application but it cannot replace a visit to the doctor. For earaches, hearing problems and other problems, seek professional medical advice.
- 3.This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- 4.WARN! DO NOT use this product for children under 3 years old.
- 5.Store the otoscope out of the reach of children. Do not allow children to use the device unattended. Children may injure themselves when using the otoscope without supervision.
- 6.The product is not a toy. Do not let Your children to play with it!
- 7.Do not share the otoscope among individuals.
- 8.When used, the temperature of the earwax removal device is close to the human body temperature.
- 9.Attention! Note that during use the temperature of the device will rise slightly (the maximum will not exceed 40°C), which does not affect the human body or the operation of the otoscope.
- 10.Be very careful when cleaning earwax with the nozzles. Do not insert them too deeply into the ear canal.
- 11.Use this product only as intended and described in this manual. Do not place any accessories, which aren't approved by the manufacturer and do not make any improvements or changes on the product, as this may lead to its improper work or damage.
- 12.Do not try to modify the product, to repair it by yourself when damaged or to replace any of the parts with such that are not provided from the manufacturer/ importer.
- 13.The otoscope is not waterproof. Avoid direct contact with water or liquids.
- 14.Never drop the device. Avoid hard impact or dropping the device.
- 15.Do not place the device where water or any other liquid can drip or splash onto it.
- 16.Wipe the lens gently with a medical alcohol swab when cleaning.
- 17.Be sure to clean the earwax remover attachment every time after using it.
- 18.The device must always storage in clean and dry environment.
- 19.Take care not to spill food, water or any other liquid on the power cord.
- 20.Do not use the device if it is not working normally, if it is damaged or has been dropped in water. Contact the shop from which you bought the product or the importer, in order to remove the damage.

CHARGING THE DEVICE

The device is with a built-in rechargeable battery with capacity of 300mAh. The battery is charged with the included USB-C cable from a standard charger with USB output (not included) or from a USB port on a computer.

The full charging of the battery is for 60 minutes. After that you can use the unit for approximately 90 minutes without it to be connected to the charger. To charge the battery use only the supplied cable.

WARNING! CHARGE THE RECEIVER COMPLETELY FOR 60 MINUTES BEFORE FIRST USE OF THE PRODUCT.

IMPORTANT INFORMATION FOR THE BATTERY OF THE RECEIVER (PARENT UNIT)

1. Before first time use charge the battery completely.
2. The product is with a built-in Li-ion battery 3.7V/ 4000 mAh.
3. The built-in rechargeable battery cannot be taken out of the product.

BATTERY CHARGING SHOULD ONLY BE DONE BY AN ADULT!

WARNING!

- Always use only the USB cable provided with the bought product. **NEVER USE** the USB cable of another brand or another product.
- Children are not allowed to handle the adapter. Only an adult must handle it. After charging you must always store it at a place out of the reach of children. The adapter is not a toy.
- Before you start charging the product you must check whether the adapter, battery and cables are in good working order. If there is damage on any of these parts, you must stop using them, until the damage is removed.
- Avoid short-circuiting.
- Store at room temperature.
- Do not expose to direct sunlight.
- **IMPORTANT!** The battery must **ONLY** be replaced at an authorized service centre.
- The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. **DO NOT** throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

FUNCTIONS AND FEATURES OF THE PRODUCT

The product has many features:

- Easy to set up, connects to your home Wi-Fi network.
- Free app –CooLeer, Complies with Android, iOS.
- High quality real-time images from inside the ear.
- Built-in lithium-ion battery that allows up to 90 minutes of operation on a full charge.
- Slim camera diameter - 3.5 mm.
- Ability to create videos and photos that are saved directly to the smartphone.
- Ability to rotate the camera image. Built-in 5-axis gyroscope can ensure stable and smooth image transmission even when rotating and maintain stable and accurate video direction.
- 6 LED lights at the end of the camera to improve visualization during use.
- Suitable for adults and children - 8 different shaped attachments that adapt to different ages or ear sizes.

TECHNICAL SPECIFICATIONS

1. Input Type/Current: DC 5V/0.5A.
2. Connecting via: Wi-Fi 2.4 GHz.
3. Web Standards: 802.11g/802.11b (2.400-2.500 GHz).
4. Application: CooLeer, compatible with Android, iOS.
5. 3 MP HD Camera, 30 fps, CMOS sensor.
6. Lens Diameter: 3,5 mm.
7. Battery Capacity: 300 mAh
8. Charging time: 60 minutes.
9. Working time: 90 minutes of continuous operation.
10. Focal Length: 1,5 – 2 cm.
11. Depth of Field: 1 – 5 cm.

10. Focal Length: 1,5 – 2 cm.
11. Depth of Field: 1 – 5 cm.
12. Temperature limits when using the device: -10°C / 45°C.
13. Internal antenna.
14. Gyroscope: 5-axis.

BEFORE SET-UP AND USE OF THE PRODUCT

1. Before setting up the product check:

- If you have access to the Wi-Fi network (2.4 GHz Wi-Fi).
 - If you are close near to the Wi-Fi router, to gain a better Wi-Fi signal.
 - If you use Wi-Fi repeater, make sure that your router and the Wi-Fi repeater have the same SSID and that you use the same password for the router and repeater.
 - You have the password at your hand. Always connect the otoscope to a secure network.
 - Make sure that your smart device runs on iOS and Android.
2. Download the CooLeer app from App Store or Google Play.

3. Or Scan the QR code:

DOWNLOADING THE APP COOLEER AND CONNECTING TO THE DEVICE

Follow the instructions and sequence for installing the app and connecting your mobile device to the otoscope as indicated in images 1 to 6.

1. Scan the QR-Code to download the App CooLeer – **Figure 1.**
2. The app is available for both iOS and Android and can be downloaded from Google Play or App Store to your mobile phone. – **Figure 2.**
3. Open the APP to add the device Digital otoscope – **Figure 3.**
4. Turn on the otoscope to complete the connection – **Figure 4.** Press and hold the unit power button (A) for 2 seconds, the camera indicator (B) starts flashing blue and the lens lights up.

Note: The otoscope supports only 2.4G Wi-Fi. Before adding the device make sure that your router is with 2.4G Wi-Fi and your mobile device is connected to the 2.4G Wi-Fi network of your home. Make sure the otoscope battery is fully charged.

5. Connect the smartphone to the product's Wi-Fi. Enter the application and press the button to switch to operating mode - **Figure 5.**

6. To ensure normal use of the otoscope, the following basic settings must be made – **Figure 6:**

6.1 Do you allow the CooLeer app to use your device location ?

(Used to provide geographic location- based features or services)

Possible options:

- Allowed when using.
- Allowed for this use only.
- Reject.

ALWAYS SELECT: Allowed when using.

6.2 "CooLeer-XXXXX" cannot access the Internet, can the Wi-Fi Assistant be allowed to switch to a better network?

Possible options:

- Not allowed.
- Allow.

ALWAYS SELECT: Not allowed.

CONTROL BUTTONS AND ICONS – FIGURE A

1. **Lens light:** 6 LED lights to improve visualization during use
2. **Zoom IN and OUT:** Press and hold the Zoom button for 2 seconds to operate zoom in/out. Press the ZOOM button again, the image will return to normal. The default setting is the image to be in normal scale.
3. **Screen rotation:** Rotate the camera image.
4. **Mirror the image:** Screen mirroring that facilitates viewing areas that are difficult to reach.
5. **Take photo.**
6. **Take video.**
7. **Album.**
8. **Screen Lock.**
9. **Remaining battery:** Battery status.

1. The camera is blurry/unclear.

Wipe the lens gently with an alcohol swab.

2. No response when booting.

Try again after fully charging. Check the Wi-Fi signal.

3. The App crashes.

Clean up the running APPs of the phone, and then Restart the APP.

4. The camera light does not light on.

Try restarting the product.

5. The product flashes a red light.

Low battery: the battery charge is lower than 10% - charge the battery and restart the unit.

6. After the product is successfully connected, the real-time image cannot be displayed normally.

Disconnect the phone from the product's Wi-Fi, restart the product, reopen the APP, and reconnect to the product's WIFI.

7. The Wi-Fi connection is unstable and disconnected after a few seconds.

Restart the product and reconnect to the product's Wi-Fi.

CLEANING AND MAINTENANCE

1. Do not immerse the otoscope into water and do not wash under running water.
2. Do not use cleansing spray and liquid cleansers.
3. Before cleaning the otoscope, you have to unplug the supply cord from the mains outlet.
4. Clean the lens carefully with a cotton swab lightly soaked in medical alcohol.
5. Clean the earwax removal scoop with water and a mild detergent each time after using it.
6. Clean the device itself with the cleaning brush included.
7. If you will not use the otoscope for a long time, store it and its supply cord in a cool and dry place.
8. If the product is damaged and cannot be used anymore, do not throw it away with the household waste but hand it in at an official collection point for recycling. By doing this, you help to preserve the environment
9. If you find any damage or missing/ broken parts, stop using the camera, until the damages are removed.
10. Do not try to repair the product by yourself. Contact the shop, from which you bought the product.

REGULATORY NOTICE

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) this device must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

-Reorient or relocate the receiving antenna.

-Increase the separation between the equipment and receiver.

-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

-Consult the dealer or an experienced radio/TV technician for help.

CE MARKING AND EO CONFORMITY

The product, Wi-Fi digital otoscope OTO (part number Y-201), is designed and manufactured in accordance with all applicable requirements of European directives. The device complies with the requirements of the Radio and Telecommunications Equipment Directive 1999/5/EU (replaced in 2017 by RED 2014/53/EU), the Electromagnetic Compatibility Directive 2004/108/EU (replaced in April 2016 by 2014/30/EU) and the Low Voltage Directive 2006/95/EU (replaced in April 2016 by 2014/35/EU) issued by the Commission of the European Community.

The product complies with the requirements of the following harmonised standards:

EN 55014-1:2021; EN IEC 61000-3-2:2019/A1:2021; EN 61000-3-3:2013+A2:2021; EN 55014-2:2021; IEC 62321-4:2013+AMD1:2017; IEC 62321-5:2013; IEC 62321-6:2015; IEC 62321-7-1:2015; IEC 62321-7-2:2017; IEC 62321-8:2017.

The complete text of EC declaration for compliance can be found at the following Internet address: www.moni.bg.

According to the European Waste Management Directive 2012/19 / EU, this symbol on the product or on its packaging means that it must not be treated as household waste and must be disposed of in the designated recycling area for electronic equipment. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. For more detailed information about recycling of this product, please contact your local Civic Office or your household waste disposal service.

Made for CANGAROO; Manufacturer and Importer: Moni Trade LTD

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Email: office@moni.bg

EN WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions. The warranty shall not apply in the case of damage caused by improper use, wear or accidental events.

For the duration of the warranty on conformity defects please refer to the specific provisions of applicable national standards in the country of purchase, where provided.

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